

## Introducing the AirSport<sup>™</sup> Ankle Brace

Designed for the athlete or chronic sufferer of ankle sprains



**The AirSport Ankle Brace is easy to use**


**STEP IN.**



**STRAP UP.**



**AND GO!**

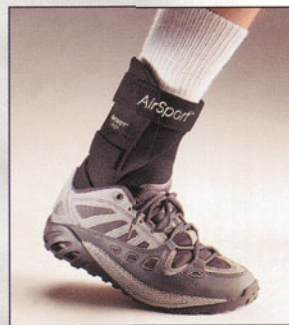




# Aircast® AirSport™ Ankle Brace


## Innovative "Step-In" Design Simplifies Application



Put on sock and slide foot, toes first, into back of brace.

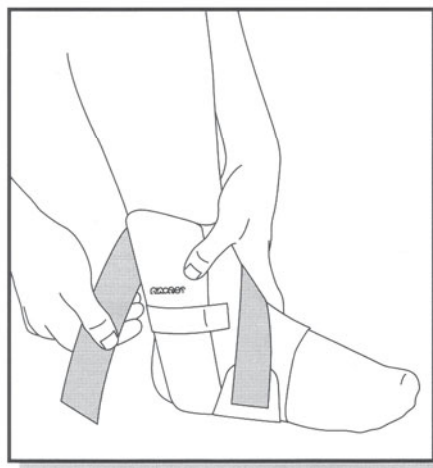

Align shells with sides of ankle.  
Secure Achilles strap and  
ATF cross-strap.

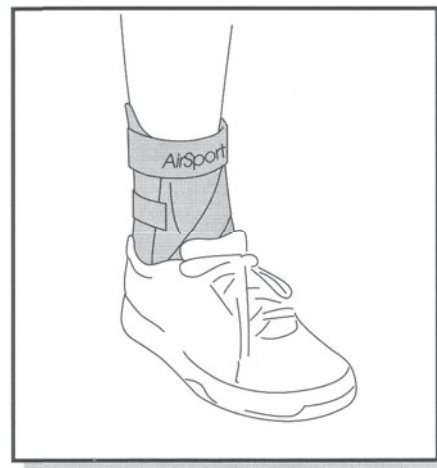

Put on lace shoe or sneaker.

Available in three sizes, left or right models:

**Small**  
02MS (L/R)

**Medium**  
02MM (L/R)

**Large**  
02ML (L/R)

This sizing chart can be used as a guide (sizes may overlap):

		Shoe Size				
		Men: up to 7	7½-9	9½-11	11½-13	13½+
		Women: 5-8½	9-11½	11-12½	12½+	—
Foot Width	Narrow:	Sm	Sm	Med	Med	Lrg
	Regular:	Sm	Med	Med	Lrg	Lrg
	Wde:	Med	Med	Lrg	Lrg	Lrg

### LATEX

All Aircast products are latex-free.

### CARE

The entire AirSport can be hand washed in lukewarm water with mild soap and air dried.

### WARRANTY POLICY

Aircast will provide prompt refund for any product that does not satisfy the customer for a period of 90 days from purchase.

### DJO Asia-Pacific

Rm. 701, 7/F, Tower 1, Grand Central Plaza, Shatin, Hong Kong

crystal.chan@djortho.com

+852 310 522 37 (Office) +852 310 514 44 (Fax) +852 904 825 57 (Mobile)

www.djortho.com © 2007 DJO Asia-Pacific

### Exclusive Distributor:

Syarikat Wellchem Sdn. Bhd.

928 - 929, Jalan 17/38, 46400 Petaling Jaya,  
Selangor D.E., Malaysia.

Tel: (03) 7957 2449

Email: mgnt@wellchem.com

Website: www.wellchem.com



NEVER STOP GETTING BETTER®

R 8/1/00  
04328A