ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:
Aids in brace Suspension.

APPLICATION INFORMATION:

- Unfasten the upper calf strap(s) and remove the strap pads from both straps.
- Fold the strap ends on your brace in half so end is attached (Velcro®) to itself.
- Orient the Anti-Migration Band so one belt loop is to the front and two are to the back. ACL/CI Versions: Feed the anterior strap end through the belt loop located on the opposite end of the hook (rough) velcro on the Anti-Migration Band. Note: for PCL version skip note 3.
- Feed the posterior strap through the middle belt loop and then the adjacent loop located next to the hook (rough) velcro on the Anti-Migration Band.
- Once the Anti-Migration Band has been applied, follow your standard recommended strapping sequence. The Anti-Migration Band will fit against the skin. Adjust and close it so it fits comfortably.

Note: For ACL/CI Versions, it is not necessary to un-attach the posterior calf strap when applying your brace, loosen before applying.

CLEANING INSTRUCTIONS: To clean liners and condyle pads, hand wash in 30°C / 85°F cold water using mild soap, rinse thoroughly, and air dry. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARRANTY: DJO, LLC will replace the unit for material or workmanship defects for a period of six months from the date of sale.

WARNINGS AND PRECAUTIONS:

This product is to be used under the supervision of a medical professional. If you experience any pain, swelling or sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

PATENT PENDING. FOR SINGLE PATIENT USE ONLY.

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